EN

 READ THE INSTRUCTION MANUAL

# Cacicol®

## Matrix therapy for regeneration

### Corneal healing

### Eye drops solution

**INGREDIENTS:**

Poly (carboxymethyl)glucose sulfate), Dextran T40, sodium chloride, purified water.

The box contains 5 unit doses of 0.33 mL of eye drops solution, packed in an aluminium sachet.

Preservative-free solution.

STERILE	A
---------	---

**WHEN TO USE CACICOL:**

**CACICOL®** is intended for the management of chronic corneal wound healing, such as persistent epithelial defects and persistent anterior corneal dystrophies with associated pain.

**Use CACICOL® with a medical prescription.**

**PROPERTIES:**

**CACICOL®** is an RGTA® (ReGeneraTing Agents) that improves both the speed and quality of tissue healing. It starts the natural process to allow new tissue to grow, the cells that support the tissue to heal, and eases pain.

**HOW TO USE CACICOL®:****METHOD AND ROUTE OF ADMINISTRATION:****USE ONLY IN THE EYE**

- Wash your hands well before use.
- Open the aluminium sachet.
- Take one unit dose container from the strip and carefully close the aluminium sachet.
- Open the unit dose container.
- Drop one or two drops into the eye to be treated while looking up and pulling the lower eyelid down slightly.
- Do not touch the tip of the container on the eye or eyelids.
- Throw away the unit dose container after use. Do not keep it to use again.

**DOSE:**

Use <b>CACICOL®</b> exactly as your doctor has told you.
--

Use one to two drops of **CACICOL®** once a week. Your ophthalmologist may change your dose.

If you use **CACICOL®** more often than you are told to, the healing effect may not be as good.

If you need to use any other eye drops as well as **CACICOL®**, use **CACICOL®** first and then wait at least 15 minutes before using the other eye drops.

**Tell your ophthalmologist if you are using or have recently used any other eye products, including any you got without a prescription.**

**SPECIAL WARNINGS AND PRECAUTIONS FOR USE:**

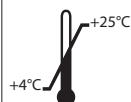
- Do not use if you are less than 18 years old.
- Do not use if you are pregnant or breast-feeding.
- Use exactly as your ophthalmologist has told you to.
- Do not use if you have an eye infection (red, sore, watery eye).
- Do not use if you are already using antibiotics for the eye belonging to the aminoglycosides such as neomycin or gentamicin.
- Do not use with solutions containing silver salts, iodine, or copper (as these can bind to the ingredients).
- Do not use if you are allergic to any of the ingredients.
- Do not use damaged containers.
- Do not use any tools to open containers.
- Do not touch the surface of the eye with the tip of the container.
- Do not inject, do not swallow.

Keep out of the sight and reach of children.

**POSSIBLE SIDE EFFECTS:**

If you are sensitive to heparin or similar drugs, you could feel a reaction on the eye when you use **CACICOL®**. This should not stop the healing. If the reaction is too strong, stop **CACICOL®** and tell your ophthalmologist as soon as possible.

If you get any side effects, talk to your ophthalmologist or pharmacist or to the manufacturer.

**HOW TO STORE CACICOL®:**

Store between 4 and 25°C.

Keep the unopened single-dose containers in the aluminium sachet to protect them from direct light. **DO NOT USE AFTER THE EXPIRY DATE CLEARLY MARKED ON THE OUTER CARTON.** The expiry date refers to the product in unopened packaging, stored correctly.

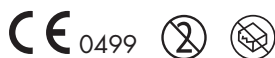
Do not use damaged unit doses.

Throw away the opened unit dose container immediately after you have used it, even if there is some solution left. Do not keep it to use again.

**MANUFACTURER:**
 OTR3 - 4 rue Française - 75001 Paris - France
**DISTRIBUTOR:**

**Spectrum Thea Pharmaceuticals Limited**  
MED IC3  
Keele University Science and Business Park  
Keele  
Staffordshire  
ST5 5NP  
Phone number: 0870 192 3283

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